

## ndd EasyOne Spirometer Configuration Settings

- 1) Turn the machine on. "Easy One" will appear on the screen and the machine will buzz.
- 2) Locate the MAIN menu.  
It will say "MAIN" on the left side of the screen, going from top to bottom.  
Use forward arrow to scroll down to CONFIGURATION. Hit enter. Top of list is TEST SETTINGS. Hit enter to select.
  - a. Hit enter to scroll to INTERPRETATION and change NLHEP to \_\_\_\_ (blank). Hit enter.
  - b. Hit enter eight times to move to STORAGE, use forward arrow to change to 3 BEST and hit enter. (*You will hear a tone at this point.*)
- 3) Use forward arrow to move to GENERAL SETTINGS, hit enter.
  - a. Hit enter to move to ALPHA ID. Change to YES using forward arrow, hit enter.
  - b. Set TECH ID to YES using forward arrow, hit enter.
  - c. Hit enter three times to move to AGE/BIRTH, hit forward arrow to select BIRTH. Hit enter.
- 4) Hit enter six times till you return to Configuration menu. (*You will hear a tone at this point.*)
- 5) Use forward arrow to move down to REPORT SETTINGS. Hit enter.
  - a. Hit enter to move to DATA. Use forward arrow to change to 3 Best Data. Hit enter.
  - b. This takes you to CURVE. Use forward arrow to change to 3 Best. Hit enter.
  - c. Hit enter five more times to exit Report Settings. (*You will hear a tone at this point.*)

In general, you use the arrows to change the selection and the enter button to accept the change. The 0 key is also the escape key.