

EasyWare Data Upload User Guide

Sync the data on your spirometer with EasyWare on your PC


- Connect the purple cradle to your computer using the USB cord
- Turn the spirometer on and place it in the cradle
- Data synchronization will start automatically

Upload the data to be over-read


Using Internet Explorer,
go to: imtr-uw.org
Click SITE LOGIN.




Enter your assigned Site ID and Password.
Click LOG IN.

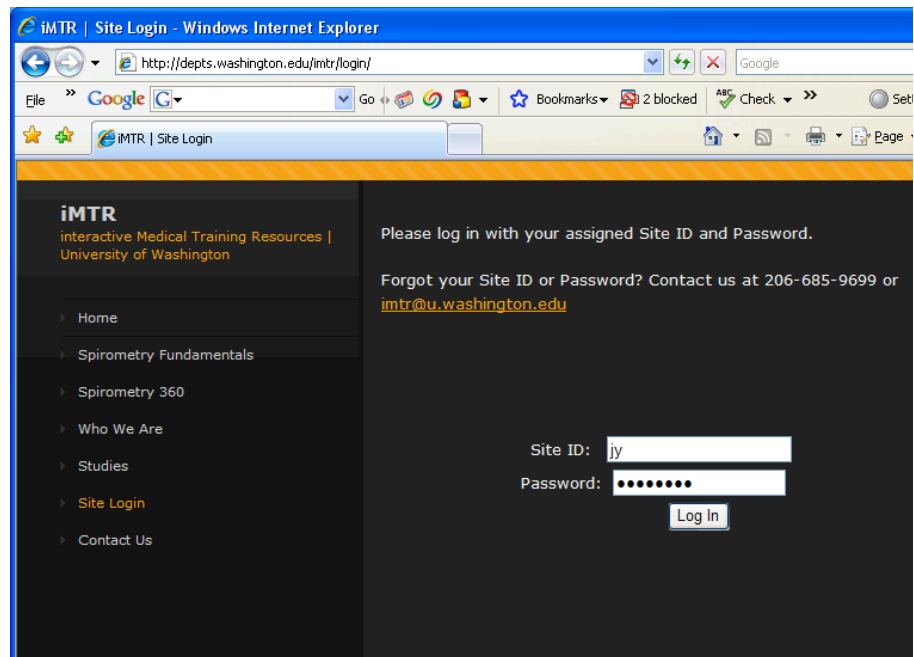


Click CONTINUE

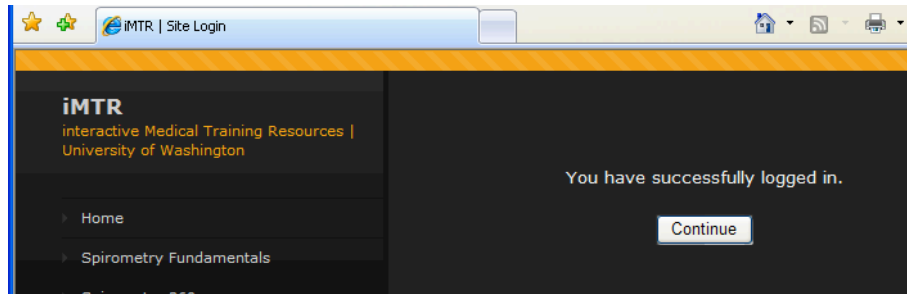

Follow the 4 steps.

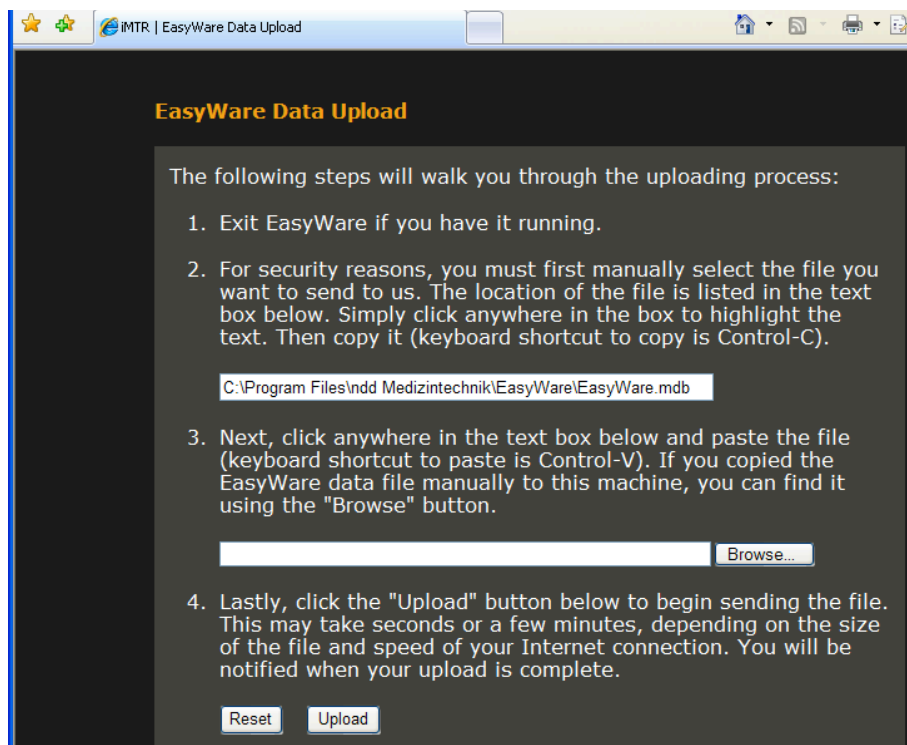

You will receive a confirmation when your Data Upload is complete.

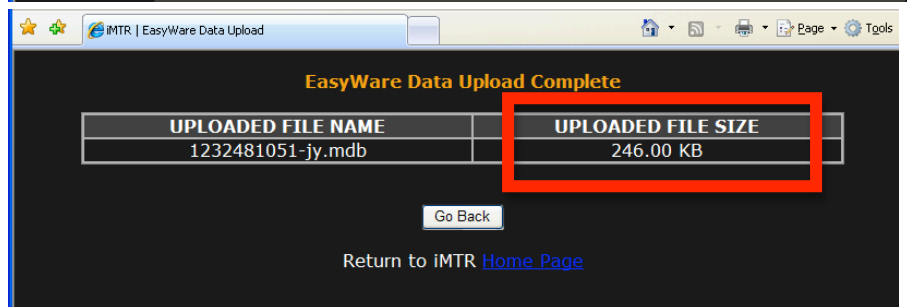

If your Upload failed, click GO BACK to try again.

