

MACH7 Cardiology Job Aide:

Link to application:


(Clinical Studio) <http://telerad.uwmedicine.org/ClinicalStudio/>

Processing images to PACS:

- 1) Log in to Clinical Studio with your AMC credentials.



- 2) When you first log in you will need to configure your default worklist. Click on Stand Alone on which worklist you want to add, enter a View and Description and click on Save. Fileroom WL is configured to show all incoming images from outside hospitals.

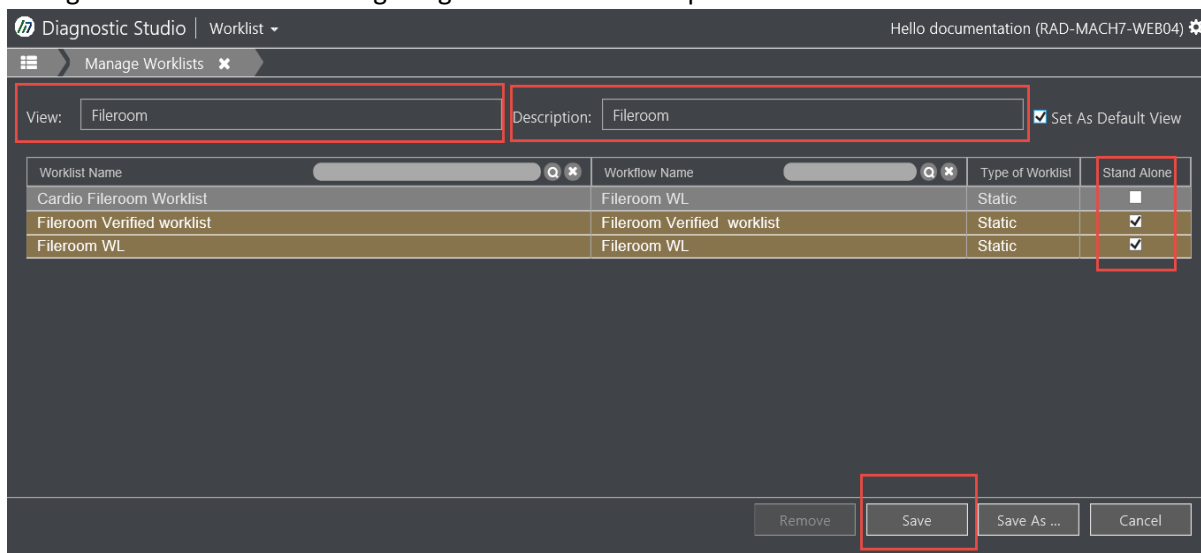


- 3) To process images to ISCV (Cardiology PACS) find your patient on the Fileroom worklist and open up the exam you want to process to PACS.

Fileroom WL (5) (0) (0)											
Institution	Patient Name	MRN	DOB	Referred By	Gender	Study Description	Modalities	Accession	Procedure Date	Department	Status
EMIX	Zztest*Test	AA9072239347	11/11/1911	AA9072239347	F		MG	040220101415	4/2/2010 2:06 PM		New
Ketchikan Gener...	ZZTEST*LISA*W	02593163	5/11/1964	02593163	F	MAMM STEREO...	MG	100243587	4/29/2016 10:35 ...		New
Peace Health So...	ZZTEST*BERTHA*W	02692957	4/5/1985	02692957	F	XR KNEE BILAT	CR	100004556	10/15/2021 5:25 ...	X.RAY	New
SeattleCancerCa...	ZZTEST*SCQC TW...	U5171352	12/22/1990	U5171352	F	Unspecified CT	CT		10/26/2021 6:07 ...		New
PLAZA ORTHO	ZZTEST*ALEX*A	21006633	3/7/1975	21006633	M	XR ABDOMEN 1...	CR	41968412	11/4/2021 6:53 AM		New

4) Right Click on image- Export- To DICOM Entity- Study.

*You can also send parts of the exam by Image or Series.

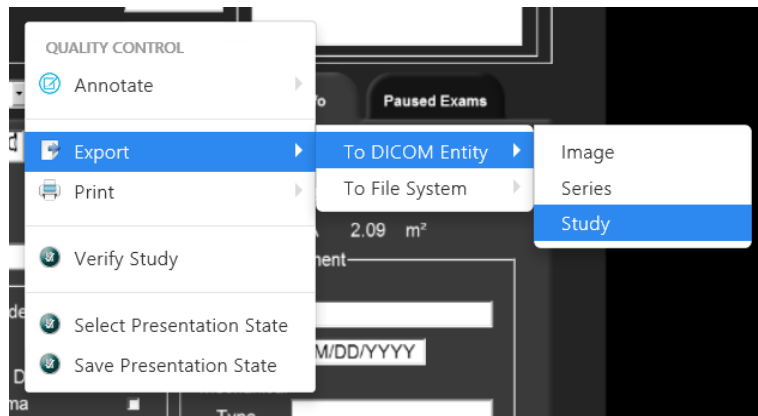

5) Use UWTELERAD as your Export Source Entity Name and choose your PACS destination in Export Destination Entity Name

A screenshot of a form with two fields. The first field is labeled "Export Source Entity Name:" and contains the text "UWTELERAD". The second field is labeled "Export Destination Entity Name:" and contains the text "_2 - ISCV (CARDIOLOGY PACS)". Both fields have a magnifying glass icon on the right side.

6) Click on Export

Export Source Entity Name:
UWTELERAD

Export Destination Entity Name:
_2 - ISCV (CARDIOLOGY PACS)

Export Cancel