

Send to UW Medicine

(Using Expedo) *Sender's Guide*

Instructions for the Outside Hospital to Send to UW directly from their PACS system (using Expedo).

PACS -> Cloud -> UW

Cloud-Based Image Exchange Solution



lifeIMAGE®

We see care differently.

We believe timely access to medical information saves lives and money. It also helps doctors make better decisions. The lifeIMAGE network connects physicians with patients' imaging history and patients with specialists, anywhere.

Please enter your login information to access your account.

Email:

Password:

[Forgot your password?](#)

Exams available for 30 days

Referral Box (Using Expedo) is for Other Hospitals to Send to UW Medicine from their PACS.

This method requires the Sending Hospital to have:

- **Expedo Application installed as a DICOM router**
- **lifeIMAGE Referral Box User Account (see page 3)**
- **Web Form Hospital Key (see page 4)**

CONTACT UW Helpdesk to put in a request:

(206) 543-7012 mcsos@uw.edu

lifeIMAGE Referral Box DICOM router Install Request




For More Information, visit the UW Medicine Imaging Services website:

<http://depts.washington.edu/pacshelp/docs/Training/lifeimage.html>

1. Send Exam(s)

Once you send from PACS, it is instantly available!

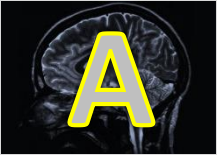

Centricity Login

User Name: YourNetID

Password: *****


Server: UWMC

Login Clear Quit



All Exams

Patient Name	Patient ID	LastFour	Procedure	Mdl	Img	Study Time	Online	Accession	Stat
zztest							y		
ZZTEST, BINKY	U3104813	4813	CT THERAPY PLANNING ...	CT	1	2011.02.17 11:44:36	Y	6566755	90
ZZTEST, BINKY	U3104813	4813	XR ABDOMEN 1 VIEW	CR	5	2011.06.10 09:45:30	Y	6739817	55
ZZTEST, BINKY					56	2011.07.25 08:02:44	Y	6747859	55
ZZTEST, BINKY					66	2011.11.30 11:36:13	Y	6892893	10
ZZTEST, BINKY					3	2009.04.26 02:47:18	Y	7156861	10
ZZTEST, BINKY	U3104813		4D ...	CT	240	2013.02.08 14:08:41	Y	7606320	55
ZZTEST, BOGUSDI					4	2002.08.04 20:54:11	Y		55
ZZTEST, BOGUSDI					58	2012.06.22 11:57:38	Y		55
ZZTEST, CONN					62	2011.01.06 15:24:01	Y		55

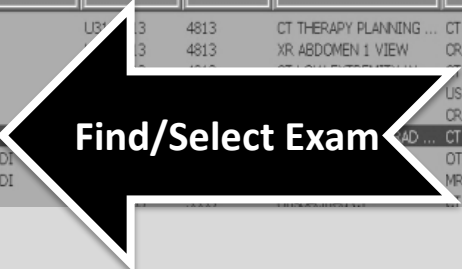

All Exams

Patient Name	Patient ID	LastFour	Procedure	Mdl	Img	Study Time	Online	Ac
zztest							y	
ZZTEST, BINKY	U3104813							65
ZZTEST, BINKY	U3104813							67
ZZTEST, BINKY	U3104813							67
ZZTEST, BINKY	U3104813							68
ZZTEST, BINKY	U3104813							71
ZZTEST, BINKY	U3104813							76
ZZTEST, PATIENT	U2390747							56
ZZTEST, PATIENT	U2390747							13
ZZTEST, PATIENT	U2390747							56
ZZTEST, PATIENT	U2390747							56
ZZTEST, PATIENT	U2390747							50
ZZTEST, PATIENT	U2390747							55
ZZTEST, PATIENT	U2390747							55
ZZTEST, PATIENT	U2390747							56
ZZTEST, PATIENT	U2390747							58
ZZTEST, PATIENT	U2390747							58
ZZTEST, PATIENT	U2390747							25
ZZTEST, PATIENT	U2390747							27

Exam Export

DICOM Send

Please Select Item: All Images in Exam(s)

Please Select Destination: LifeIMAGE Test Outbox

Send Mode: Immediate Batch

Reasons for Sending Exam(s): Archive Consultation Read Archive, Read Transcribe Archive, Transcribe Exam Data Update Exam Complete

Send Cancel



2. CHECK STATUS (Optional)

Verify it uploaded to UW Medicine & Check status

Verify Exam Upload Completion

A


Go to url: <https://cloud.lifeimage.com/>

Login to lifeIMAGE



B

How to Verify that the exams are there?

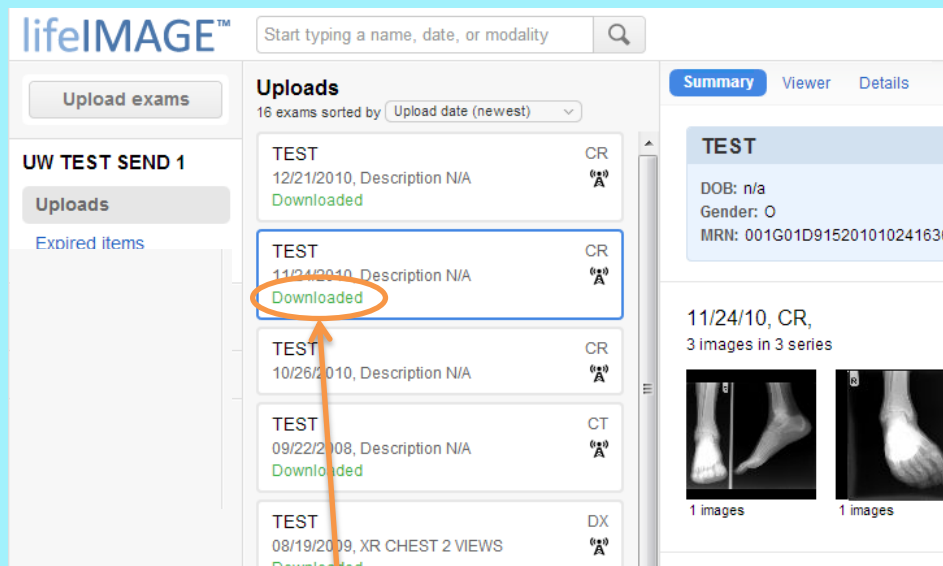

If you see “Downloaded”,
Your exam upload is complete.
UW Medicine can view the exam.




DONE!

3. Submit Web Form

This will notify UW Medicine to obtain uploaded exam

Go to url: www.reachuw.org


A

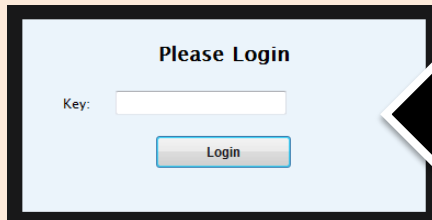

Login with
your **key**

B

Complete form &
Click **Submit Request**

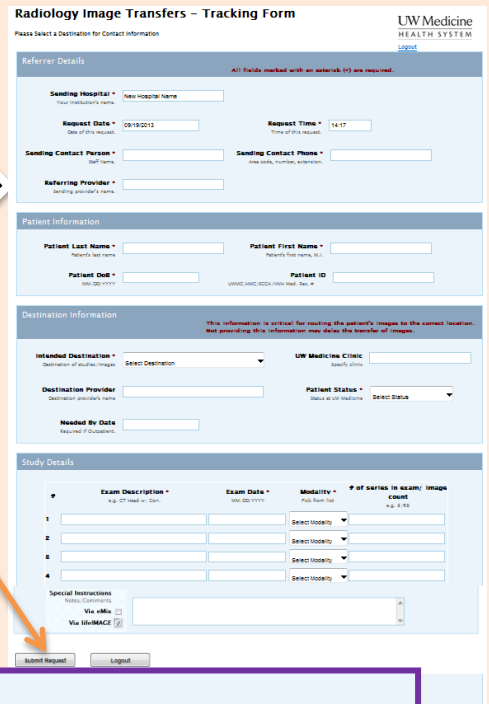

Remember to
CHECK This Box
before Submitting!

Special Instructions
Notes/Comments

Via eMix

Via lifeIMAGE

Q?

Don't have a web form **key** for your Hospital?

Contact: Radiology Fileroom UWMC – Arthur Gines artgines@uw.edu
Radiology HMC - George gcu@uw.edu OR Jeff farrj@uw.edu

CHECK STATUS

Check download status at <https://cloud.lifeimage.com/>

The screenshot shows the Life Image Hospital cloud interface. At the top left is the Life Image Hospital logo. The top right shows the user is logged in as 'TEST AAUser' with links for 'Help', 'Account settings', and 'Log out'. The main content is divided into several sections:

- Uploading exams:** A progress bar shows 16.1MB of 33.4MB uploaded. A 'Cancel' button is present.
- OVERREADS:** A section for overreads.
- TEST GROUP:** A section for test groups, with 'Uploads' and 'Purged' options.
- Uploads:** A list of 4 exams sorted by 'newest upload':
 - TEST^BINKY (MR):** 2/09/2000 | PFIZER STROKE. Status: **Uploading**.
 - TEST^BINKY (CR):** 9/09/2002 | ABDOMEN, KUB. Status: **Rejected by Receiver**.
 - HADDAD WILLIE (CR):** 04/26/2009, KNEE RT TRAUMA 4 VIEWS ... Status: **Downloaded**.
 - HADDAD WILLIE (CR):** 04/26/2009, KNEE RT TRAUMA 4 VIEWS ... Status: **Ready**.
- Summary:** Includes 'DropBox' and 'Viewer' tabs.
- Message:** 'This exam is still being uploaded. The information is subject to change.'
- Warning:** '30 days remaining until this exam is purged from the DropBox.'
- Patient:** TEST^BINKY (F), DOB: Jan 01, 1969.
- Sending hospital:** TEST Hospital.
- Primary contact:** John Doe, Phone: 206 598-3300, Email: johndoe@u.washington.edu.
- Notes:** Need your second opinion.

All Exams uploaded to the Referral Box will be available for only 30 days.

EXAM STATUS LIST

Uploading – Exam uploading to the Referral Box

Ready – Exam Uploaded successfully. Receiver can view or download exam

Downloaded – Exam downloaded by the Receiver

Rejected by Receiver – Exam was rejected, Click to see reason and message

EXAM INFO/VIEWER

Exam information and the Built-in Viewer is available

LI Life Image HOSPITAL

Welcome, TEST AAUser
[Help](#) | [Account settings](#) | [Log out](#)

Upload exams

OVERREADS

TEST GROUP

Uploads

Purged


Uploads

4 exams sorted by newest upload

TEST^BINKY	MR
2/09/2000 PFIZER STROKE	
Waiting for receiver to download	
TEST^BINKY	CR
9/09/2002 ABDOMEN,KUB	
Rejected by Receiver	
TEST^BINKY	CR

PFIZER STROKE

2/09/2000, 539 images in 6 series

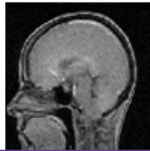

- PATIENT DEMOGRAPHICS
- EXAM HISTORY
- EXAM INFORMATION

Summary **DropBox** Viewer

25 days remaining until this exam is purged from the DropBox.

Patient
TEST^BINKY (F)
DOB: Jan 01, 1969

Sending hospital
TEST Hospital

Primary contact
John Doe
Phone: 206 598-3300
Email: johndoe@u.washington.edu

Notes
Please upload for Surgical Specialties

Exam will **PURGE** after **30** days

Summary DropBox **Viewer**

TEST^BINKY,
ABDOMEN,KUB
WL: 511 WW: 1024 (DICOM)
Lossy compression

Mon Sep 9 2002
11:25:37 GMT-0800

Viewer w/Basic PACS functions

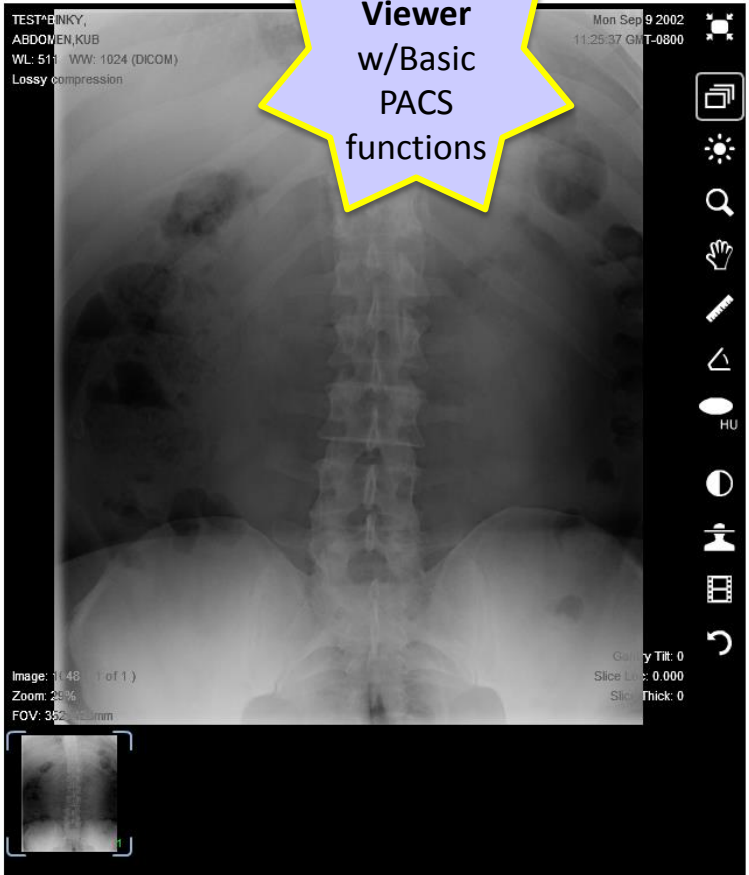


Image: 1048 (1 of 1)
Zoom: 25%
FOV: 352.00mm

Gallery Tilt: 0
Slice Loc: 0.000
Slice Thick: 0